NSK Varios Tips

VarioSurg Tips

OPERATION MANUAL

Tip Power Guide

NAKANISHI INC. www.nsk-dental.com
700 Shimohinata, Kanuma, Tochigi 322-8666, Japan

NSK Europe GmbH
Elly-Beinhorn-Str. 8, 65760 Eschborn, Germany

Visit our website

Specifications are subject to change without notice.

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EN • If the tip doesn't vibrate when the device is being activated, remove the handpiece from the patient's mouth, then restart the handpiece operation. If this does not improve the situation, the tip may have been loosened during use. Check that the tip is securely attached using the correct tool. • Use the tip ONLY in the correct mode and never exceed the maximum power defined for the tip by this manual. If used outside the prescribed power range, the Tip may break, and the tooth surface or soft tissue may be damaged. • Before connecting or disconnecting the Handpiece Cord or Irrigation Tube make sure to remove the Tip first to avoid injury by the Tip. • Always connect the sterilized handpiece. • When inserting a Tip always wear surgical gloves and ensure the Tip, Handpiece and Tip Wrench are in a hygienic condition. • If debris is found on the Tip thread clean the Tip immediately so as to avoid possible weak vibration. • Do not use a Tip which has been damaged, bent or corroded as it will be weakened and may break while in use.

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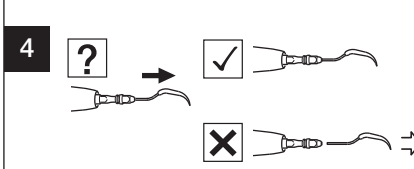
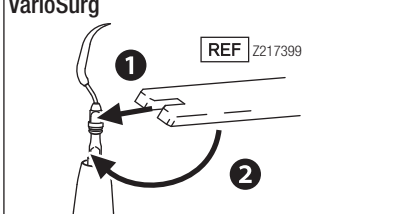
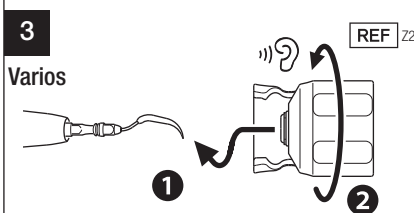
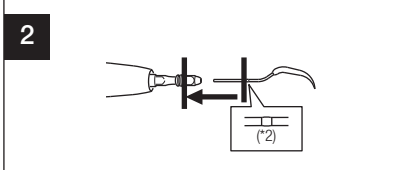
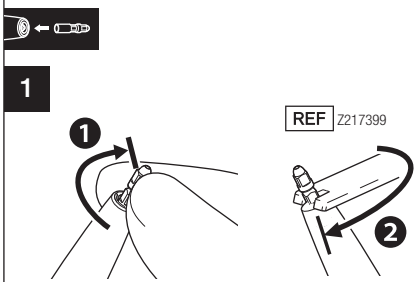
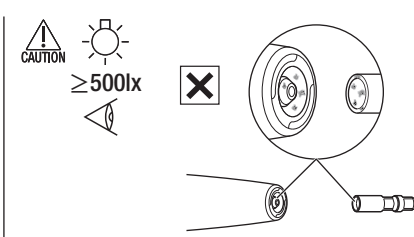
EN Immediately after a treatment, the tip should be cleaned, disinfected, sterilized and stored in accordance with this operation manual.

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EN Failure to properly maintain as below, the tip may cause infection, product failure, overheating leading to burn injuries:
- Be sure to clean the tip immediately (within 30 minutes) after use to remove residue.
- Be sure to disinfect the tip and the tip wrench, within 2 hours after cleaning.

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EN • Before using an alcohol based product for internal disinfection, the Tip must first be cleaned with a non-protein-fixing product (e.g. ALPRO WL-clean). • Use only cleaning/disinfectant products listed in this manual. • Follow any additional local directives, standards and guidelines for cleaning, disinfection, and sterilization.

• Do not clean, immerse, or wipe with electrolyzed-oxidizing water (strong acidic water, super acidic water), strong acidic and strong alkaline liquid medicine, solvent containing chlorine, benzene, or thinner. • Use clean and dry compressed air for cleaning. • Tips must be removed from the washer-disinfector immediately (within 1 hour) after the cleaning, disinfecting and drying cycle is complete to prevent corrosion.

• After using washer-disinfector, dry the product completely. The remaining moisture causes internal corrosion etc. • Follow local rules, regulations and guidelines regarding the reprocessing of devices. • Do not autoclave the tip and the tip wrench with other instruments even when it is in a pouch. This is to prevent possible discoloration and damage to the product from chemical residue on other instruments. • Do not heat or cool the product too quickly. Rapid change in temperature could cause damage to the product. • Be sure to use sterilizers that can perform sterilization up to 135°C. In some sterilizers, the chamber temperature may exceed 135°C. Do not use these sterilizers as failure of the product could occur. Contact the sterilizer manufacturer for detailed information about cycle temperatures. • Clean the tip and the tip wrench prior to disinfection. If blood remains on the internal surface it can become clotted and cause product failure.

• Immediately after sterilization is complete (within 1 hour), remove the tip and the tip wrench from the sterilizer. Failure to do so may cause corrosion.

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



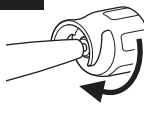
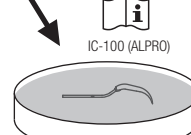
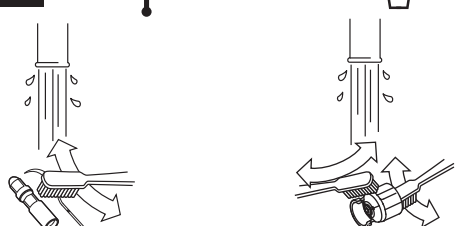
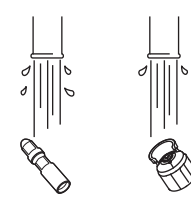
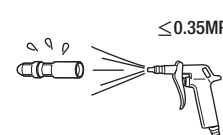
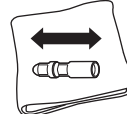
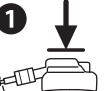
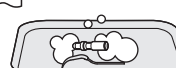
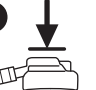
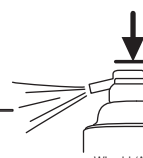
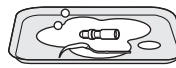
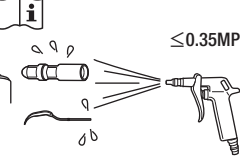
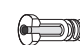
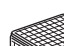
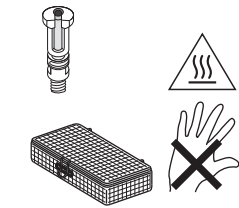
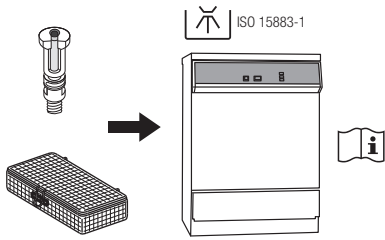
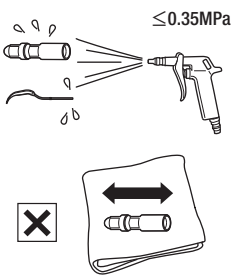
4

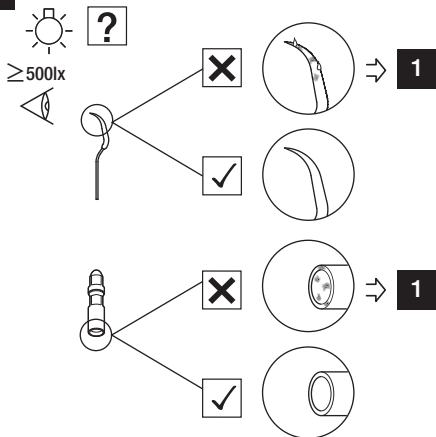1

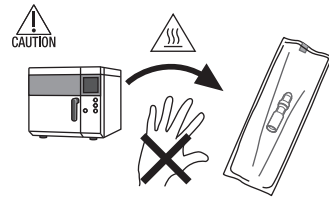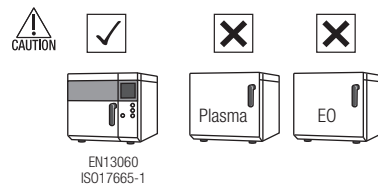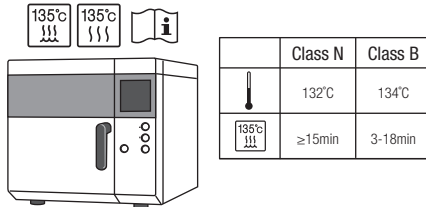EN • Do not use sterilization pouches containing water-soluble adhesive components such as PVA (polyvinyl alcohol). Adhesive components washed out during sterilization may enter the product, causing problems such as malfunction, sticking and failure to operate correctly. • Do not place Tips in the Tip Holder, as these Tips are too long. Use an autoclave pouch when sterilizing Maintenance Tips.

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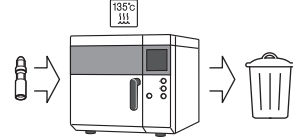
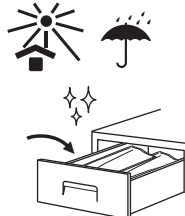
1

EN When your medical device has reached the end of its service life, it must be disposed of via a specialist collection, removal, recycling or destruction channel. Prior to doing so, in order to avoid the health risks to operators handling the disposal of medical equipment, as well as the risks of environmental contamination caused thereof, you need to confirm the equipment is decontaminated. Therefore contact your nearest NSK dealer, or one of the NSK branches to find out how to proceed.

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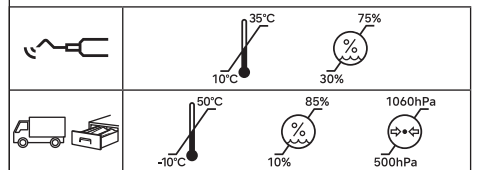
EN • This tip is specially designed for the Ultrasonic Systems manufactured by NSK. Do not use with other systems. • Be sure to always carefully read the operation manual for the Ultrasonic System before use to understand the usage and performance of the instrument. • Be sure to always use with sufficient coolant water. Insufficient coolant water could cause a burn of the affected area or the overheating of the instrument. • Be sure to always use the tip at a setting not exceeding the maximum power level. If the tip is used in excess of the maximum power, the tip could break or excess cutting could occur. • Do not touch the top part of the tip to avoid an injury.

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EN • This product must only be used by a qualified personnel in accordance with the indications for use and the directions for use stated in the operation manual of the Ultrasonic System it is attached to and this operation manual. Do not use it for any other purpose. Operators and all others in the area must wear eye protection and a mask when operating this tip. • Do not use a Tip which has been damaged, bent or corroded as it will be weakened and may break while in use. • If debris is found on the Tip through clean the Tip immediately so as to avoid possible weak vibration. • Do not use a Tip which has been damaged, bent or corroded as it will be weakened and may break while in use. • Prior to clinical use, inspect the tip outside the patient's oral cavity. Check for vibration, noise and overheating. If any abnormalities are found, stop using the tip immediately and contact your Authorized NSK Dealer. • The tip is a consumable item and wears with use. If it is worn, abnormal noise, weak or no vibration, or damage may occur. Replace the tip with a new one when it is worn. Also, the tip can break due to metal fatigue. Replace with a new tip on a regular basis. • Tip Wrench will wear with frequent use and may need annual replacement. • The tip may be discolored by repeated processing. If the tip is discolored significantly, replace it with a new one. • Do not grind or bend the tip. It may break or may not vibrate during use. • Keep away from patients with cardiac pacemakers. • The tip is delivered in a non-sterile condition and must be autoclaved prior use. • Always connect the sterilized handpiece. • For irrigation, avoid use of ozonated water, oxidizing water (strongly acidic water, superacidic water, electrolyzed water, etc.), or hypochlorous solution. Doing so may lead to glooming, corrosion of metal parts, etc. • Perform suction while using the tip, since the tip may break depending on its use condition. • Even if the power level is correctly set for the Tip in use, application of excessive pressure to the Tip may damage it. • During vibration, do not hold the tip nor make contact with the tip for an extended time. • If there are abnormalities in water supply during operation, stop using the tip immediately and replace it with a new one. • If the tip cracks or breaks during use, immediately stop using the tip, and replace with a new tip. Operation with a damaged tip could injure the gums, periodontal tissue, or other locations. • Do not apply more pressure than required to the tip end. This could cause the tip to break or bend. • Do not place the active Tip anywhere other than the specific treatment area, as the ultrasonic vibration may cause oral injury. • Start with a low power setting within the power range, observing the patient's reaction and according to the treatment plan. • Before use, check that the tip is firmly attached to the handpiece. • Before connecting or disconnecting the Handpiece Cord or Irrigation Tube make sure to remove the Tip first to avoid injury by the Tip. • Refer to the operation manual of the ultrasonic system for EMC information. • Report all serious events related to the product immediately to the manufacturer and responsible national authority.

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