




POWER GUIDE

Varios 370
Varios 970
Varios Combi Pro

iPiezo engine® Varios2

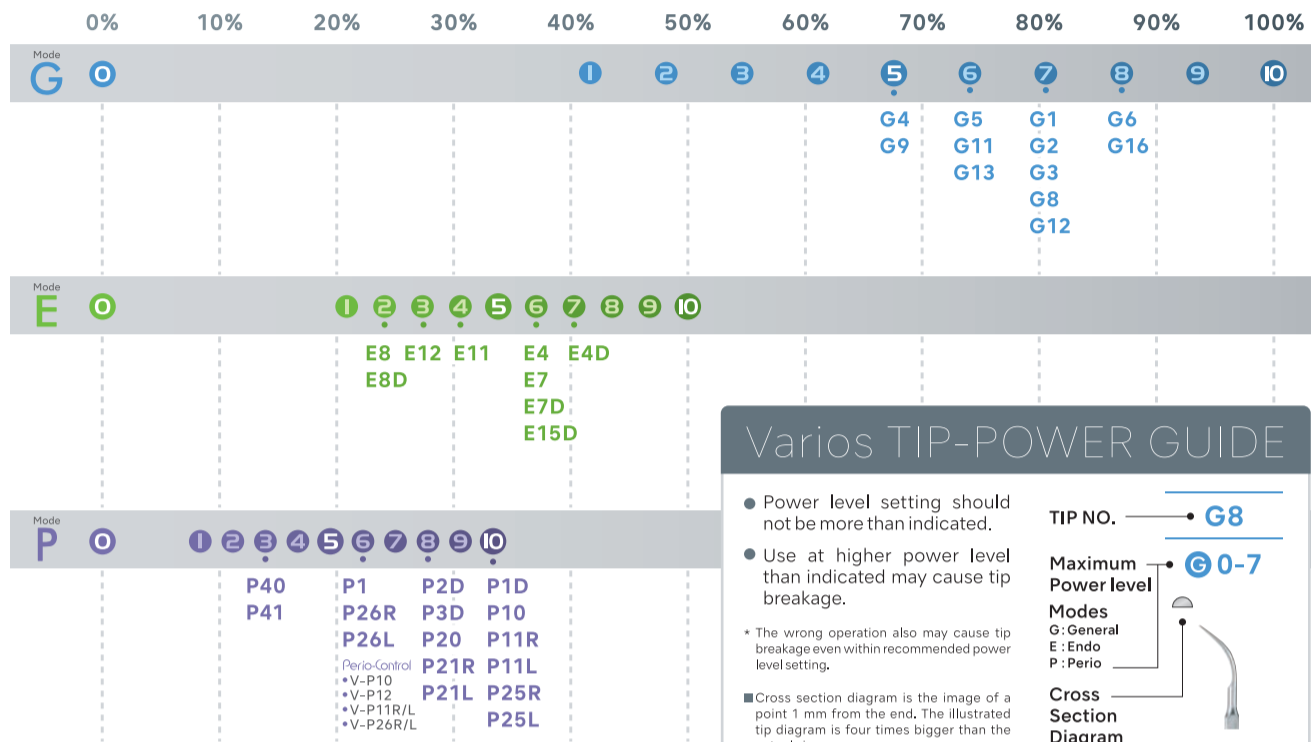

Varios TIP-POWER GUIDE

- Power level setting should not be more than indicated.
- Use at higher power level than indicated may cause tip breakage.
- * The wrong operation also may cause tip breakage even within recommended power level setting.
- Cross section diagram is the image of a point 1 mm from the end. The illustrated tip diagram is four times bigger than the actual size.

TIP NO. → G8

Maximum Power level → G0-7

Modes
G: General
E: Endo
P: Perio

Cross Section Diagram

R0000043 000 2018.09 01 ©

